



ZULU SCOOTER

USER MANUAL



Congratulations and thank you for choosing the
Aspire Zulu Scooter.

This user manual contains a description of the product
and important guidelines to ensure correct and safe use.
It is important to read this manual carefully prior to use.
It is especially important to read and follow the
safety requirements.

Aidacare continuously improves products and reserves
the right to change the specifications and functions of
products without notice.

**If you have any queries, please contact your dealer or
Aidacare directly. Contact information is located
on the last page of this manual.**

© Aidacare Limited 2026

No part of this user manual may be copied, distributed, reproduced, scanned, or stored on any
electronic database, whether in whole or in part in any form or by any means.

Due care has been taken to ensure all the information contained in this user manual is correct at
the time of printing. All measurements, pictures, colours, and weight capacities are to be used as
a guide only. We reserve the right to modify the design or appearance of any product displayed in
this user manual without prior notice.

TABLE OF CONTENTS

1. Intended Use	4
2. Product Description	4
3. Symbols Used In This User Manual	4
4. Symbols Used on This Product	5
5. Warnings & General Safety Information	6
6. List of Components	18
7. Technical Specifications	19
8. Control Panel	20
9. Battery & Charging the Battery	32
10. Care, Maintenance & Customisation	37
11. Troubleshooting	41
12. Warranty	44
13. Compliance	44
14. Recycling Information	45
15. Manufacturer, Importer & EU Representative	46

1. INTENDED USE

The Aspire Zulu Scooter is a motorised electric scooter for use outdoors primarily on flat surfaces such as pavements, roads, parking lots, and drive ways. It is intended to increase the mobility of people who are both physically and cognitively capable of correctly assessing driving situations and reacting accordingly to them at any time. This product should be used as a tool to assist with mobility or walking difficulties. This product is developed for indoor and limited outdoor use. When used outdoors, the scooter should remain on sealed, level terrain (examples include, but are not limited to, shopping centres, medical centres, flat and level footpaths, and environments with accessibility-focused layouts). Any other use outside of the intended purpose is not advised.

2. PRODUCT DESCRIPTION

The Aspire Zulu Scooter is a four wheeled large mobility scooter. The product is made from steel, so it is durable and strong. This scooter offers great maneuverability, stability, and comfort, which features four independent suspensions. It has a generously padded seat with integrated suspension and backrest with 15" wheels for longer rides. This scooter is powered with two motors fitted to the rear wheels.

3. SYMBOLS USED IN THIS USER MANUAL

The symbols below are used throughout this user manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.



WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a white triangle with a black border.



ALWAYS! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white infinity symbol on a black dot with a white border.



DO NOT! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol with a black circle and black slash.

4. SYMBOLS USED ON THIS PRODUCT



Stickers shown are not to scale.

	Product Code		Driving Speed		Not Crash Tested
	Serial Number		Climbing Angle		Read User Manual
	Manufacturer		Safe Working Load		Caution
	Country of Manufacturer		Product Weight		Medical Device

5. WARNINGS & GENERAL SAFETY INFORMATION

5.1 BEFORE DRIVING FOR THE FIRST TIME

Before taking the first trip with mobility scooter, you should familiarise yourself well the operation of mobility scooter and with the operation elements. Please take your time to read this introduction booklet.

Before driving, please evaluate your personal condition, and fully understand the operation of mobility scooter.



DO NOT assemble, maintain, and operate the mobility scooter before you read this instruction booklet.



WARNING! Observe and obey all pedestrian rules and regulations in which you are riding.



WARNING! Mobility device may only be used on the traffic routes for which it is approved in accordance with the relevant national legislation.



ALWAYS be aware of pedestrians and situations which might require extra care when using your scooter on public walk ways and footpaths.



DO NOT drive your scooter if you are under the influence of alcohol or medication that may affect your ability.



DO NOT try not to drive scooter at night.



ALWAYS turn the key off before getting on or off (see section 8).



ALWAYS please observe all relevant rules and regulations pertaining to pedestrians and road users, at all times when you are driving the scooter.



DO NOT turn the power on before you get in and sit securely on the seat.



WARNING! Be certain that the power is turned off when get in, get out. This will eliminate the possibility of accidentally activating the wigwag controls and causing injury.



WARNING! Keep your weight toward the middle of the deck. Putting most or all of your weight on the edge of the deck may cause an unstable condition.



WARNING! Only one person at a time can ride the Zulu Mobility Scooter. Do not carry passengers under any circumstances.

PRACTICE TIPS

If you are new to drive a scooter, it is a good idea to practice in a clear, safe space on a sound level surface, ie. park or playground.

Basic functions to practice: Wig wag accelerate / Wig wag release / Stop / Reverse / Turn / Ramp proceeding.

Set the speed control to its lowest speed, slightly increase the speed when you are getting familiar with the scooter. (Refer to 8.2, Speed Dial Knob)

Practice operating your scooter in the presence of an attendant. Remember that only with practice will you become a competent driver. Practice these basic functions until you feel that you have control of your scooter.

5.2 CAUTIONS WHEN DRIVING



WARNING! Please do the daily check before your journey always. (Refer to section 10.1, Daily check)



DO NOT extend your body over the mobility scooter.



WARNING! Please make sure you are safe when crossing the level crossing.



WARNING! Never place the scooter in freewheel mode when on any sort of an incline or decline. When the scooter is parked, the lever for engaging and disengaging the motors must be locked firmly into the “DRIVE” position. (Refer to section 9, lever adjustment)



WARNING! Cross the railroad crossing, make sure the wheels pass the rail at 90 degree angles, avoid scooter stuck on railroad crossing.



WARNING! When descending an incline, use the slowest speed possible. If the descent is faster than you desire, release the throttle lever to stop the scooter. Then press the throttle lever slightly to control the speed.



WARNING! Please make sure you are safe when crossing the level crossing.



WARNING! When crossing the level crossing, please be aware of the wheel and rail are perpendicular.



DO NOT use the mobile phone and wireless mobile devices while driving the scooter.



DO NOT use the batteries of mobility scooter to charge any other electric devices, except the accessories from original manufacturer.



DO NOT drive on slope over the limit. (Refer to section 7, curb climbable)



WARNING! Reduce speed when descending to prevent any danger.



DO NOT increase speed when ascending.
(Refer to section 8.2, Speed Dial Knob)



DO NOT drive to across the obstacle over the limit.
(Refer to section 5.5)



DO NOT attempt to drive the scooter in rain, wet grass, or any other potentially hazardous condition.



WARNING! Ensure that the lights of scooter are turned on while driving at night or in poor visibility.



WARNING! Please stop operating the scooter if the batteries have drained, continuous operation may damage the scooter.



ALWAYS follow the local pedestrian traffic rules when driving outside.



WARNING! When turning, reduce your speed and maintain a stable center of gravity. This greatly reduces the possibility of a tip or fall.



DO NOT turn off the power while driving.



DO NOT stretch your body out on the scooter. For maximum stability, lean forward in your body while proceeding up ramps, inclines, curbs, or any low rise.



DO NOT attempt to have your scooter climb or descend an obstacle that is inordinately high. This may cause the scooter to tip.



DO NOT attempt to use your scooter on an escalator.



DO NOT make sharp turns while driving. This may cause the scooter to tip.

5.3 GENERAL SAFETY INFORMATION FOR SCOOTER



WARNING! Batteries should be fully charged before using for maximum performance and longevity. (Refer to section 9, Battery & Charging The Battery)



WARNING! The maximum load of the scooter is 226kg. Do not exceed the maximum permissible load. Exceeding the max. weight rating may result in injury to yourself.



WARNING! The maximum load of the front basket is 3kg exceeding the max. weight may result damage to basket.



WARNING! The mobility device is only designed for use by a single occupant whose maximum weight does not exceed the maximum permissible load of the device. Never use the mobility device to transport more than one person (including children).



DO NOT attempt to carry out maintenance work that is not described in this user manual.



DO NOT change, modify, remove any parts from products especially safety protected parts such as anti-tippers.



WARNING! Completely deflate the tires (air tires) before dismantling the rim.



WARNING! Materials and assemblies of scooter is flame resistant.



WARNING! Avoid exposure the scooter to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition. Any direct contact with water can cause damage to the scooter and electrical system.



DO NOT remove the anti-tip wheels or modify your scooter in any way that is not authorised.



WARNING! Only use the chargers, accessories, or components supplied as original equipment with your scooter.



WARNING! Immediately stop using the scooter if you encounter a problem with the scooter. Turn off the power and contact Aidacare for further checking.

5.4 CAUTIONS WHEN DRIVING ON INCLINES




WARNING! The scooter has been rated to a maximum climbable height, obstacle height, and gap. (Refer to section 7, Technical Specifications)



WARNING! Never drive on a slope that exceeds the rated slope.



WARNING! The safe working load at different ramp degree (please refer to following picture).

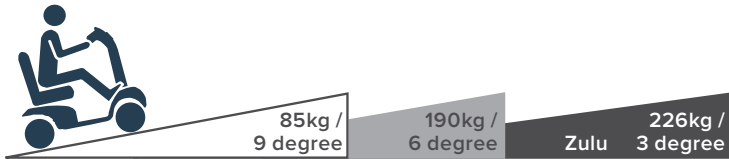

WARNING! Your scooter's ability to travel up inclines is affected by your weight, your scooter's speed, your angle of approach to the incline, and your scooter setup.



WARNING! Please avoid driving on a long ramp or any uneven terrain to prevent any damage to the motor.



WARNING! The batteries voltage normally will go up when driving on descending road. If the battery voltage becomes too high, the over-voltage protection will be activated by slowing the speed till the scooter stops. (error code :ERR3 will be displaying). Please pull over the scooter to the safe area, release the throttle, and restart the scooter again.



WARNING! When driving down a ramp or uneven terrain, keep the scooter's speed adjustment set to the slowest speed setting to ensure a safely controlled driving.



WARNING! If the speed is too fast, release the throttle control lever, let the scooter stop. When you feel that you again have control of your scooter, push the throttle control lever forward and continue safely driving.



DO NOT turn around at high speed on ascending, descending ramp.


5.5 CAUTIONS WHEN CLIMBING



WARNING! The maximum height of obstacle and curb that scooter can climb is up to 9cm with run-up (A).



WARNING! The maximum gap that scooter can drive over is 22cm (B).

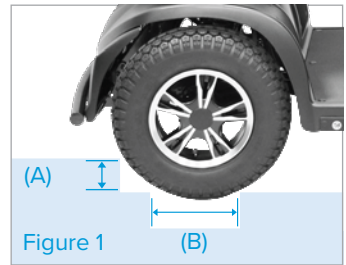

WARNING! Always overcome obstacles straight on at 90 degrees, in one stroke, do not stop halfway! Firstly, make sure the front wheels of the scooter is perpendicular to the obstacle, approach the obstacle slowly until the front wheels touch the obstacle, increase the speed and ensure that the front wheels and rear wheels have moved over the obstacles, and then you may reduce the speed.



WARNING! When driving scooter on ramp, adjust body center of gravity for scooter stability.



WARNING! Never attempt to overcome an obstacle when on an uphill or downhill gradient or it may increase the chance of tipping! Adjust the seatback into an upright position before climbing an obstacle, or it may increase the chance of tipping!

5.6 ELECTROMAGNETIC INTERFERENCE AND WARNINGS



WARNING! It is very important that you read this information regarding the possible effects of Electromagnetic Interference on your mobility scooter.

Mobility scooters may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and mobile phones. The interference (from radio wave sources) can cause the mobility scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the mobility scooter control system. The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each mobility scooter can resist EMI up to certain intensity. This is called its "immunity level." The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20V/m immunity level, which would provide useful protection from the more common sources of radiated EMI. The immunity level of this mobility scooter model is not known.

There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warnings listed below, your risk to EMI will be minimised.

The sources of radiated EMI can be broadly classified into three types:

1. Hand-held portable transceivers (transmitters-receivers) with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, "walkie talkie," security, fire, and police transceivers, mobile phones, and other personal communication devices.



WARNING! Some mobile phones and similar devices transmit signals while they are ON, even when not being used.

2. Medium-range mobile transceivers, such as those used in police cars, fire trucks, ambulances, and taxis. These usually have the antenna mounted on the outside of the vehicle.
3. Long-range transmitters and transceivers such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.



WARNING! Other types of hand-held devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, and cassette players, and small appliances, such as electric shavers and hair dryers, so far as we know, are not likely to cause EMI problems to your mobility scooter.

Mobility Scooter Electromagnetic Interference

Because EM energy rapidly becomes more intense as one move closer to the transmitting antenna (source), the EM fields from hand-held radio wave sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the motorised scooter control system while using these devices. This can affect mobility scooter movement and braking. Therefore, the warnings listed below are recommended to prevent possible interference with the control system of the mobility scooter.

Warnings

Electromagnetic interference (EMI) from sources such as radio and TV stations, amateur radio (HAM) transmitters, two-way radios, and mobile phones can affect mobility scooters. Following the warnings listed below should reduce the chance of unintended brake release or motorised scooter movement which could result in serious injury.

1. Do not operate hand-held transceivers (transmitters-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as mobile phones, while the mobility scooter is turned ON.
2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
3. If unintended movement or brake release occurs, turn the motorised scooter OFF as soon as it is safe.

4. Be aware that adding accessories or components, or modifying the mobility scooter, may make it more susceptible to EMI.



WARNING! Some mobile phones and similar devices transmit signals while they are ON, even when not being used.

5. Report all incidents of unintended movement or brake release to the distributor listed on the inside front cover of this manual. Note whether there is a source of EMI nearby.

Important Information

1. 20 volts per meter (V/m) is a generally achievable and useful immunity level against EMI (as of May 1994). The higher the level, the greater the protection.
2. The immunity level of this product is at least 20/Vm.

5.7 SAFETY WARNING AND INSTRUCTION LABELS



Figure 2

1 Warning Sticker


1. Please read the User Manual carefully before using your scooter.
2. Keep the scooter properly maintained.
3. Do not drive the scooter on slippery surfaces or on slopes over 10°.
4. Do not drive on highways, crowded roads, or unfamiliar areas.
5. Do not turn at high speed in either forward or reverse.
6. Do not wash with water or leave scooter in a humid environment since water can damage the electric parts.
7. Always re-engage the emergency freewheel device after use.



- 2 Do not hang baggage or other objects on the tiller / tiller adjustment lever.




- 3 Tiller angle adjustment.

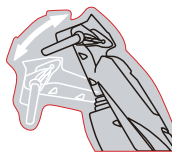

- 4 The scooter is not intended to be used as a seat in a motor vehicle.




- 5 Wiring diagram label. (open shroud to access)

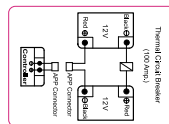

- 6 Tie-down points of the scooter. Do not sit or stay on the scooter during transporting.



- 7 N-D Lever Adjustment label which instructs freewheel mode operation. (see section 9.2, Operating The Charger, for operation)

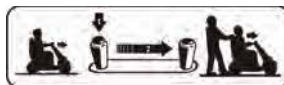

- 8 Model label.



- 9 Power Off Indication. Indicates the location to disconnect the battery.



- 10 Information Sticker states the date of manufacture and serial number of the scooter. (below the tail light)

This product has been supplied from an environmentally aware manufacturer. It may contain substances that could be harmful to the environment. Recycling must be carried out in accordance with the respective national legal provisions.



6. LIST OF COMPONENTS

1	Control Panel	
2	Front Basket	
3	Front Turning Signal	
4	Headlight	
5	Front Bumper	
6	Throttle	
7	Seat fore-aft Adjustment Lever	
8	Seat Swivel Lever	
9	Tie-down Hook	
10	Rear Mirror	
11	Tiller Angle Adjustment Handle	
12	Charging Port	
13	Front / Rear Wheels	
14	Reflector	
15	Captain Seat	
16	Adjustable Armrest (Width and height)	
17	Seatback Adjustment Handle	
18	N-D Lever	
19	Rear Turning Signal	
20	Anti-tipper	

7. TECHNICAL SPECIFICATIONS

Size	Maxi (SUCCEEDS 928)
Colour Code	Midnight Blue SCT671320
Seat Features	Slide/Swivel/Recline
Overall L x W x H	1600 x 720 x 1280mm
Seat Size D x W	510 x 560 - 610mm
Seat to Deck	460 - 510mm
Seat to Ground	670 - 720mm
Armrest Height	180 - 240mm
Min Turning Radius	1860mm
Ground Clearance	110mm
Max Climbing Angle	9°
Max Driving Speed	10km/h
Driving Range	50km
Wheels Tyres	380mm Pneumatic
Travel Ride	Full Suspension
Battery	2 x 12V 110Ah (Lead Acid)
Product Weight	178kg
Heaviest Part	N/A
Battery Weight	65.3kg
Power Motor	1000W 4 Pole
Controller	DS 160
Brake System	Electro-Mechanical
Free Wheel Mode	Yes
Charger Size	24V 12A
SWL	226kg

8. CONTROL PANEL

8.1 IDENTIFICATION OF CONTROL PANEL

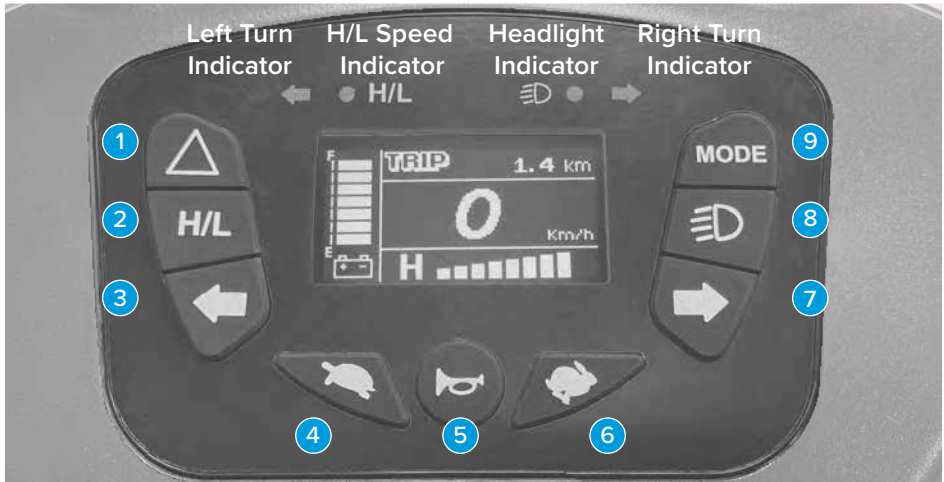



Figure 3 - Zulu Control Panel

Item	Function
1 Hazard Light	Press for Hazard on.
2 Hi/Lo Switch	Press to switch to half of the maximum speed.
3 Left Turn Signal	Switch on for left turn.
4 Deceleration	Press to decrease speed.
5 Horn	Press for warning sound.
6 Acceleration	Press to increase speed.
7 Right Turn Signal	Switch on for right turn.
8 Headlight	Press for Headlight on.
9 Mode Setting	Press mode setting button to set different mode. (Refer to 8.7)

8.2 GERNERAL OPERATION

Main Key Switch

- Turn key to the **RIGHT** - scooter on.
- Turn key to the **LEFT** - scooter off.
- Turn on the scooter, control panel will be in standby mode.

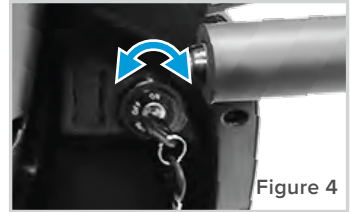

ALWAYS ensure that the scooter is switched off before getting on or off the scooter and before removing any items of the scooter.



WARNING! Turning the scooter OFF whilst driving will bring the scooter to an abrupt stop and danger.

Sleep Mode

- Scooter will go into Sleep Mode with one long beep warning sound if no throttle activity is detected for programmable time period. (Default programmable time is 30 mins.)
- When scooter is in Sleep Mode, the LCD display will be off automatically and the scooter will not respond to commands. Turn on the scooter, control panel will be in standby mode.
- To wake up the scooter, turn the power (key) off and then on again.

Speed Adjustment

The Rabbit means fast and Turtle means slow.

- Press button to increase or decrease speed.



DO NOT set the speed at maximum when operating the scooter. Adjust the speed at maximum when inclining a slope, and adjust to minimum speed when declining.

Turning Signal

- Press left side to turn on left turning signal.
- Press right side to turn on right turning signal. (Figure 6)

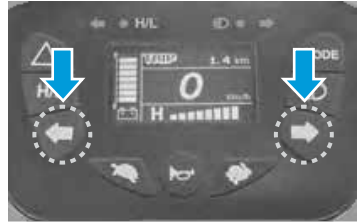



Figure 6

Headlight

Press the button  to turn on /off the headlight. (Figure 7)

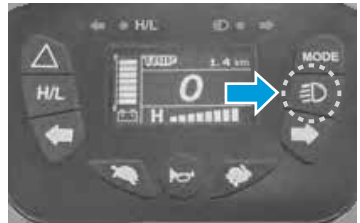


Figure 7

Horn

Press the horn button to sound buzzer. (Figure 8)

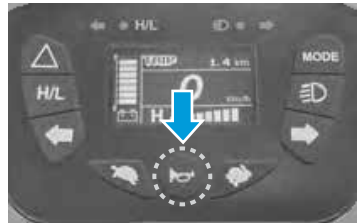


Figure 8

Throttle

- Pull the right throttle to move scooter forward. Pull the left throttle to move scooter backward. This can be reversed if required by local dealer. (Figure 9)
- These are also your accelerator. The further you depress them, the faster you go. (Subject to the position of speed control)
- Releasing both left/right throttles automatically operates the brakes to slow down and stop.



Figure 9

Electronic Brake

Press the brake button and release the throttle to stop the vehicle. Pressing the brake button while holding the throttle may cause skidding. Be careful to avoid danger. (Figure 10)

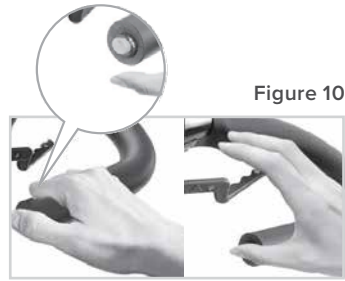


Figure 10



DO NOT pull both right and left throttle at the same time, you might not be able to control the scooter.



ALWAYS ensure the scooter is switched off before getting on or off the scooter.

Tiller Angle Adjustment

Pull the tiller adjustment downwards to adjust tiller's angle and release to lock at a desired comfortable position. (Figure 11)

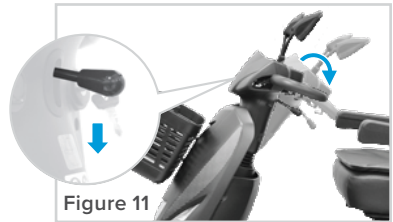


Figure 11

Seat Swivel Adjustment

Pull the swivel lever upwards to rotate seat left and right. (Figure 12)

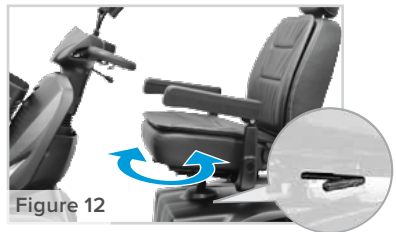


Figure 12

Seat fore-aft Adjustment

Set at a comfortable position by lifting lever forward to adjust the seat. (Figure 13)

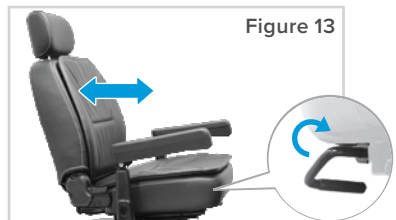


Figure 13

Seat fore-aft Adjustment

Pull the lever (Figure 14) upwards to adjust backrest's angle, then release the lever when adjusted to required position.

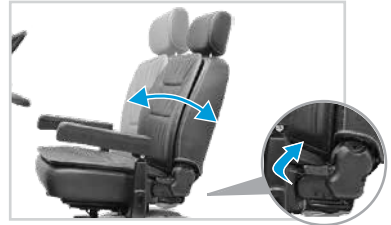


Figure 14

Armrest angle, height, and width adjustment

- **Angle:** Rotate the screw as picture to adjust the most comfortable armrest angle. (Figure 15)
- **Width:** Loosen the thumbscrews as picture to adjust the width, tighten again to lock at a desired position. (Figure 16)
- **Height:** Loosen screw, adjust to height hole, then tighten. (Figure 17)
- Pull the armrest up when getting on or off the scooter.

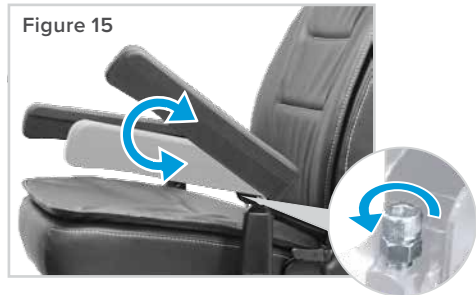


Figure 15

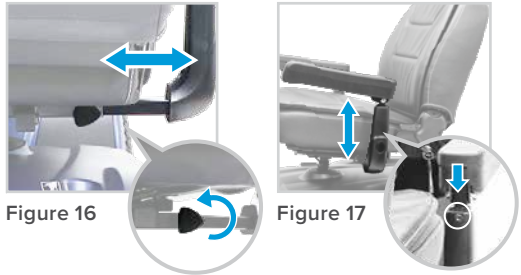


Figure 16


Figure 17



WARNING! When driving the scooter, set the seat at foremost position to prevent tip over. **DO NOT** hang heavy goods on the armrests.

8.3 FREE-WHEELING (N-D LEVER)

When scooter stopped or malfunction, press the unlocking knob on the N-D lever (X). Push the N-D lever forwards this will allow you to push the scooter by hand. (Figure 18)

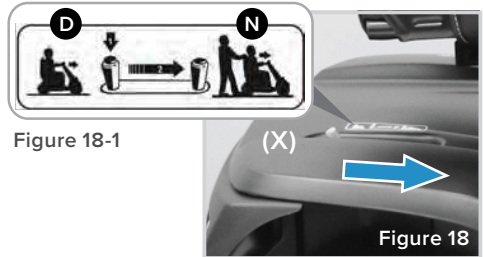


WARNING! Freewheel operation is only recommended on flat surfaces, never on slopes. Never leave your scooter on a slope with its engine switched off. Always switch the engine on immediately after pushing the scooter.



WARNING! The scooter won't operate if it is set at (N) position, to restore to its normal status, you must switch the power off and adjust to (D) position, then switch the power on.

8.4 TIE-DOWN HOOKS

- To ensure safety during the transportation. There are four fixed hooks for fixing scooter on other vehicles. (Figure 19)
- Make sure the scooter's N-D lever is in D position when transporting. (Figure 18-1)
- Do not sit on the scooter during transportation.

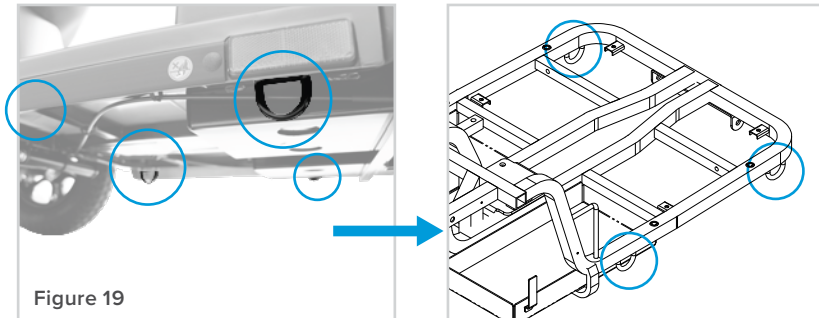


8.5 DISASSEMBLING YOUR SCOOTER

Disassembling seat: By lifting lever to remove seat upwards. (Figure 20)




Disassembling front basket: Lift the front basket upwards. (Figure 21)

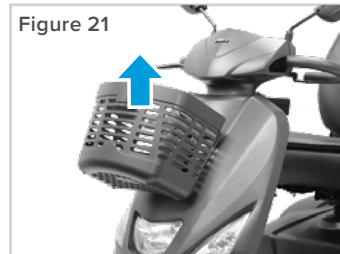

Rear compartment cover: Open the compartment cover upwards according to the **steps**.

1. Shift the gear to the “N” position. (Figure 18-1)
2. Disassembling seat: By lifting lever to remove seat upwards. (Figure 20)
3. Carefully lift and remove the rear compartment cover (A). (Figure 22)

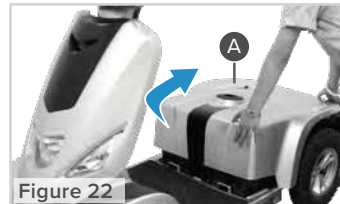

DO NOT turn on the scooter during disassembly or assembly. Keep the scooter in the “N” position to make disassembly easier and prevent parts from getting stuck.






DO NOT modify or change the scooter with non-authorized parts or accessories.

8.6 DISASSEMBLING BATTERIES

Release the battery velcro straps and disconnect the battery connectors to remove the batteries.

1. Remove rear shroud (A) upwards. (Figure 24)
2. Unplug one battery connector (B). (Figure 25 / Sticker refer to 5.7)
3. Release Velcro (C). (Figure 26)
4. Remove two batteries (D). (Figure 27)

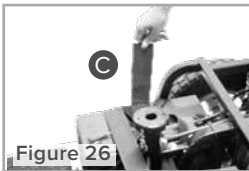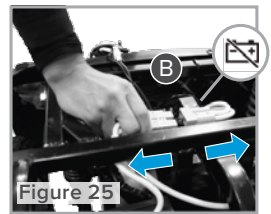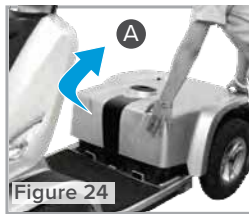

WARNING! Batteries are heavy, be careful when removing the batteries.



WARNING! Do not connect battery terminal [+] [-] with any metals to avoid danger.



WARNING! Connect the red terminal to red, blue terminal to blue.



WARNING! When replacing the batteries remove the screws and connectors in sequence and tighten each component back.



WARNING! Maintenance & repair must only be carried out by a competent engineer or authorised dealer.

8.7 [MODE] SELECTION

Press **MODE** to change display mode: **TIME**, **TEMP**, **ODO**, and **TRIP**.

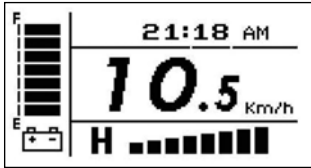


Figure 29 [TIME]

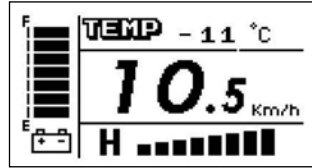


Figure 30 [TEMP]

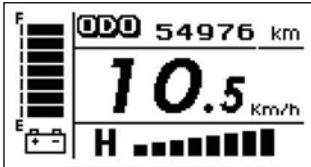


Figure 31 [ODO]

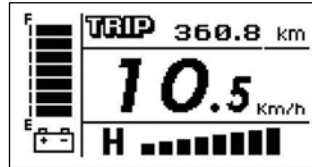






Figure 32 [TRIP]



Exit / Save:

- Setup mode will automatically quit after 15 secs without pressing **left turn** and **right turn**, and scooter will save the setting and return to operation mode.
- User can also press any buttons, except  /  / **left turn** / **right turn** to save the setting automatically and return to operation mode.

[TIME] Time mode (Figure 29):

- Hold both buttons  /  together for 2 secs to set up the time.
- Press **left turn** to increase digits / Press **right turn** to decrease digits.
- 12hr-mode : 12:00AM~11:59PM / 24hr-mode : 00:00~23:59. Tolerance: +/- 2 seconds/day.



[TEMP] Temperature mode (Figure 30):

- Press  : Centigrade. Range -20°C~50°C.
- Press  : Fahrenheit. Range -4°F~122°F.



[ODO] Odometer mode (Figure 31):


- Range: 0~99999.
- The odometer will be reset automatically to zero after the total distance reaching 99999km (62149 mile).
- The odometer (hrs) will stop counting after reaching 99999hrs.

[TRIP] Trip mode (Figure 32):

- Hold  /  together for 2 secs to reset TRIP meter.
- Range : 0.0~999.9hrs. The trip meter will stop counting when reaching 999.9hrs.

8.8 SETTING-USER DEFINE

1. Make sure the scooter (key) is off.
2. Press and hold both  /  then turn on the key. Keep holding the buttons until the LCD screen is on.
3. Press **MODE** button to select the modes you want to set up.

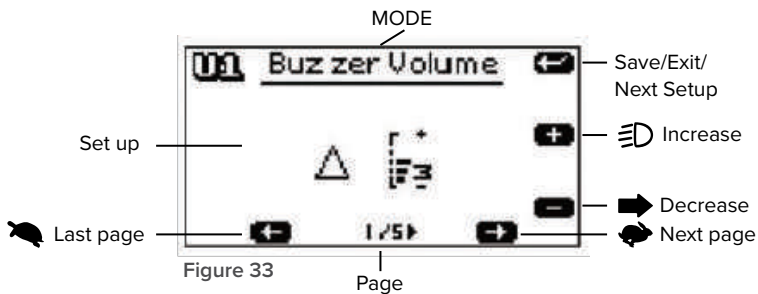

Exit /Save the setting:

Setup mode will automatically quit after 15 secs without pressing any button. Scooter will save the setting and return to operation mode.

Buzzer Volume setup (Figure 34):

Low battery warning sound is controlled by the controller.

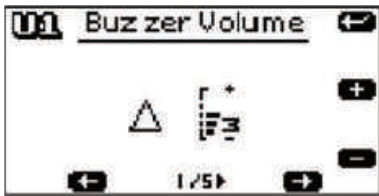


Figure 34

Item	Description	Initial value
1/5	△ Hazard Warning volume	3
2/5	↔ Indicator warning volume	3
3/5	Ⓜ Reverse warning volume	3
4/5	🔋 Low battery warning volume	off
5/5	📢 Horn volume	5

Unit setup (Figure 35):



Figure 35

Item	Description	Initial value
1/4	Clock 24 , 12	24
2/4	Speed km/h , mph	km/h
3/4	Odo km , hrs	km
4/4	Temp °C , °F	°C

LCD panel backlight (Figure 36):



Figure 36

8.9 MAINTENANCE REMINDER

- User can set the system to track distance intervals for maintenance.
- The maintenance warning light on the LCD screen will be flashing when the maintenance is due at set distance. The flashing light will be off automatically after a minute.



WARNING! The scooter/control panel can still be operating when the maintenance warning light is flashing.



WARNING! Please contact the authorised dealer for routine maintenance when the maintenance warning light is flashing.

Setting

Please reset for next maintenance distance when the maintenance is performed.

1. Turn the power (key) off.
2. Press **MODE** button and **←** button at the same time.
3. Hold the two buttons and then turn the power to get in Maintenance mode setting.
4. Press **↵** to increase digits (+ 500km/per press) or **→** to decrease digits (-500km/per press).
5. Press and hold **↻** for 2 seconds to erase the setting.
6. Press **MODE** to save and exit setup mode when finished.

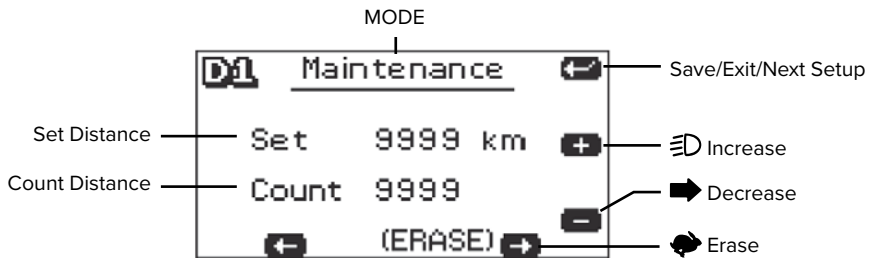


Figure 37

- The maintenance warning light will flash for a second every time when the scooter turns on if the Set Distance reaches Count Distance.
- Setup mode will automatically off after 15 secs without pressing any button. Scooter will save the setting and return to operation mode.

9. BATTERY & CHARGING THE BATTERY

9.1 BATTERY GAUGE

There is a meter which shows the batteries capacity status.

When battery gauge light trends to F, it indicates sufficient power capacity.

When battery gauge light trends to E, it indicates insufficient power capacity.

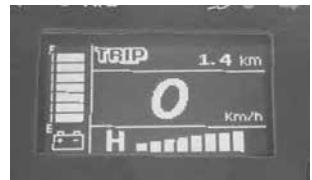


Figure 38


Status Display							
Capacity (%)	>80	<80	<65	<50	<35	<25	<20

Low Battery Warning:

- Warning Condition: The low battery warning will be activated once the battery level is less 25% of max capacity.
- Warning Sound: Beep-Beep-Beep.
- Warning Display: Flashing bar symbol.

Battery Charging Status Indication:

- After 10 seconds without pressing any key, LCD back light will be automatically off. Press any key to turn on the back light.
- Please refer to the indicators on battery charger for more accurate charging status.
- The charging status will be displayed on LCD screen while charging. (The scooter can not be operated while it is in charging.)
- Recharge the scooter immediately when the battery level drops to 20% as the remaining power can only keep scooter going for buffering 3km or less.

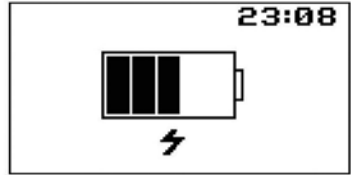

WARNING! Each country may supply different charger. The charging procedure may be different from below. If you require more details, please contact your authorised dealer.

9.2 OPERATING THE CHARGER

Batteries must be charged before using the scooter for the first time and should be recharged after each day use. You will need the scooter and the battery charger.



WARNING

WARNING! Each country may supply different charger. The charging procedure may be different from below. If you require more details, please contact your authorised dealer.



WARNING

WARNING! Be sure the scooter key is in the OFF position.



WARNING

WARNING! Do not use extension cord to connect the charger.



WARNING

WARNING! Do not use other brand charger to charge the scooter.

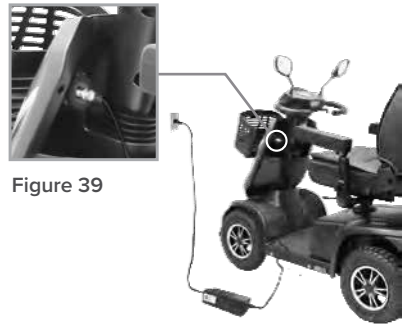


Figure 39

1. Insert battery charger cord into the charger connector on the charger output. Refer to above figure for correct position.
2. Plug the other end of the battery charger cable into a standard electrical outlet.
3. Turn the power on. Normally, the LED (Power) Light will turn on when electric current passes.
4. Charging starts. During charging, LED (Charge) will indicate orange light, when it turns to green light, that means well-charged.
5. **LED indication**
 - **Power on:** On/off LED - Yellow light on
 - **Charge on:**
 - Red light on = Charging
 - Green light on = Fully charged
 - Red flashed = Standby
 - Red flash-rapid = Fault



Figure 40

6. Charger Trouble Shooting

(A) If LED (POWER) light is off

- Check the input voltage (115V/230V) is the same as you adjust.
- If light is still off, please check and repair the battery charger.

(B) If LED (CHARGE) light is off

- Check to see clips connection is correct.
- If the battery is fully charged, the LED (CHARGE) light will be off.
- If light is still off, the battery may be defective.

(C) If ORANGE light can turn to GREEN

- The battery can not be charged. Please check and recover it.

(D) If ORANGE light turns to GREEN immediately

- Check to see the battery is fully charged, if not, the battery may be defective. Check and recover it.



The time needed to recharge will vary depending on the depletion of the batteries. Charging for longer than necessary will harm the batteries. They can not be overcharged.

9.3 INFORMATION FOR BATTERY & CHARGER

Battery

- Only be used for 12V lead-acid battery, not used for other type of battery or other voltage.
- Do not turn the key on while scooter is in charging.
- Charge the batteries after each trip. Only use the battery charger (under 8Amps) supplied with your scooter.
- If the scooter is not used for some time batteries must be recharged once every three months at least. Make sure that batteries are fully charged, and on returning, charge them again before using scooter.
- Batteries will only give maximum performance after scooter has been used and batteries have been recharged up to 10 times.

- Do not switch off, unplug, or interrupt the recharge cycle until the charging cycle has completed. (Charger indicator turns green)
- The minimum time needed to recharge will vary up to 15hrs which depends on the depletion of the batteries. Excessive or short period charging will be detrimental to battery life.
- Please charge the scooter in 0°C ~ 40°C, out of this range might affect the performance of the charger and battery.
- For longest life, your batteries should be recharged regularly. Please recharge the batteries before they run down to 20%.

Charger

Do not leave charger plugged into your scooter with charger switched off as this may discharge your batteries.

- Inspect the battery charger before each use, make sure connectors are dry and clean. Do not attempt to use an extension cord to plug in your battery charger.
- Be aware that the battery charger case may become hot during charging. Please avoid skin contact.



WARNING! The battery charger is equipped with cooling fan. If the fan does not appear to be working correctly, please turn off the charger immediately as it may be overheated. Please contact Aidacare.



WARNING! Use the battery charger in a well-ventilated area. Do not smoke as explosive gases may be generated while charging the batteries.



WARNING! Any battery faults due to unauthorised maintenance, dismantle, misuse, or accidental damage is not covered by the manufacturer's warranty.

10. CARE, MAINTENANCE & CUSTOMISATION

10.1 DAILY CHECK

Please always check your scooter before you start your every journey.

Check point	Inspection	Ref.	What to do if the inspection is failed
N-D lever	Check for correct function.	P.25	Contact Aidacare
Horn	Check for correct function.	P.22	Contact Aidacare
Throttles	Pull the wigwag to test the scooter movement.	P.22	Contact Aidacare
Electromechanical brake and Emergency hand brake	Pull the wigwag a little bit and release it to test if brake works. If your scooter comes with emergency handbrake, please check it as well.	P.23	Contact Aidacare
Battery Gauge	Check if the battery gauge is displayed and whether it is at low power.	P.32	Contact Aidacare if battery gauge is not working. Recharge the battery immediately if low
Rear mirror (s)	Check if the parts are clean and firmly tighten to the scooter and do not wobble.	P.18	Clean up the dirt by damp cloth. Tighten the screw or clamping stem that holds the mirror(s)
Lighting	Check if all lights, such as head lights, rear lights, and turn signal are functioning correctly.	P.18	Contact Aidacare

10.2 WEEKLY CHECK

Check point	Inspection	Ref.	What to do if the inspection is failed
Speed Dial Knob	Check for correct function.	P.21	Contact Aidacare
Armrests	Check if the parts are clean and firmly tighten to the scooter and do not wobble. Tighten the screw knob that holds the armrest.	P.24	Contact Aidacare
Wheels/Tires	Inflate the tire to the correct pressure and check that 1. Drive wheels rotate without wobbling. 2. Tire tread depth is not less than 0.5mm. 3. No foreign objects in tires.	P.40	Contact Aidacare
Motor	Check for correct function.	Observe while driving	Contact Aidacare
Battery Charger	Check if the charger is functioning correctly and the batteries are charged.	P.32	Contact Aidacare

10.3 MONTHLY CHECK

Check point	Inspection	Ref.	What to do if the inspection is failed
Seat / Upholstery	Check for movement and if it's worn.	P.23	Contact Aidacare
Electronics	Check if all the battery cables and connectors are firmly tighten to the scooter.	P.27	Contact Aidacare

10.4 CLEANING YOUR SCOOTER

- Do not use any abrasive or scouring liquids for cleaning. Only use a damp cloth and gentle detergent.
- Do not use hose pipe or splash water directly onto the scooter as this may cause damage to electronics.

10.5 MAINTENANCE

- User should inspect the scooter regularly to keep scooter in good running order.
- Check if the electrical cable connectors are fully connected.
- All maintenance and repair of scooter should be done by an authorised dealer.

Seat Upholstery

Only use damp cloth and a little soap to wipe the seat. Do not use abrasive cleaners as this will damage the seat.

Storage

- Please store the scooter in a dry location. If storing the scooter in long time, please disconnect the battery terminals.
- Do not store your scooter where it will be exposed to a source of direct heat, damp, oil, acid, alkaline, or where Ozone could be possibly generated. All of the above will minimise scooter / tire cycle and shorten its lifetime.

Tire

User should inspect the tires frequently for damage, the presence of foreign bodies, unusual wear and sufficient tread depth. If replacement tires are needed, please contact the nearest dealer.


The following areas require periodic inspection:

- Tire pressure between 30-35psi
- Tread depth drops below 0.5mm

Recommended range of storage,

Temperature : $-30^{\circ}\text{C} \sim +50^{\circ}\text{C}$,

Humidity : 25% \sim 85%

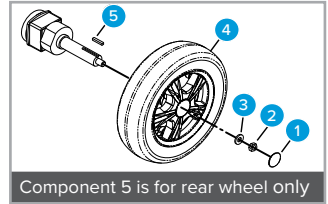

Follow these easy steps to replace the tire:

1. Turn off the scooter and remove the key. Make sure the lever is in D position before you lift the scooter.
2. Use an ratchet and socket to remove the drive wheel screw from the centre hub of the wheel. Pull the wheel off of the axle.
3. Separate the tire from the rim.
4. Remove the old tire and replace it with a new tire.
5. Slide the wheel back onto the shaft.
6. Install the drive wheel nut into the centre hub and verify the key is lined up with axle and wheel, then tighten to secure it in place. (Torque $300 \pm 30 \text{kgf-cm}$)

All maintenance and repair of scooter should be done by an authorised dealer.


11. TROUBLE SHOOTING

Here are some suggestions about solving problems you may have with your scooter. There is a self-diagnostic warning light on the control panel. To check the self-diagnostic warning light, turn on the key and count the number of blinks on the warning light.

11.1 SCOOTER WON'T MOVE WHEN KEY IS TURNED ON

Check point	Solution
Check if the power is off.	Turn the power on.
Check if the N-D lever is in Neutral position.	Switch to D (drive) position. Turn off the power and turn on again.
Check if the battery power is enough. (Battery gauge is under 25%)	Recharge the battery and then retest.
Check if the charger power cord is still plugged in scooter.	Unplug the charger power cord.

11.2 ERROR CODE

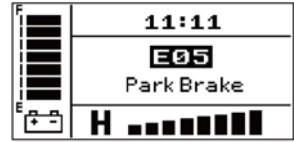

Code	Description	Initial check points
E01	Battery Low	The batteries are running low. <ul style="list-style-type: none"> • Recharge the batteries.
E02	Low Battery Fault	The batteries have run out of charge. <ul style="list-style-type: none"> • Recharge the batteries. • Check the battery and associated connections and wiring.
E03	High Battery Fault	Battery voltage is too high. This may occur if overcharged and/or traveling down a long slope. <ul style="list-style-type: none"> • If traveling down a slope, reduce your speed to minimise the amount of regenerative charging.
E04	Current limit time-out or controller overheat	The motor has been exceeding its maximum current rating for too long. <ul style="list-style-type: none"> • The scooter may have stalled. Turn the controller off, leave for a few minutes and turn back on again. • The motor may be faulty. Check the motor and associated connections and wiring.
E05	Park Brake	Either a park brake release switch is active or the park brake is faulty. <ul style="list-style-type: none"> • Check the park brake and associated connections and wiring. • Ensure any associated switches are in their correct positions.
E06	Drive Inhibit	Either a stop function is active or charger inhibits or OONAPU condition has occurred. <ul style="list-style-type: none"> • Release the stop condition (seat raised etc.) • Disconnect the battery charger. • Ensure the throttle is in Neutral when turning the controller on. • The throttle may require re-calibration.

Code	Description	Initial check points
E07	Speed Pot	The throttle, speed limit pot. SRW or their associated wiring maybe faulty. <ul style="list-style-type: none"> • Check the throttle and speed pot and associated connections and wiring.
E08	Motor Voltage	The motor or its associated wiring is faulty. <ul style="list-style-type: none"> • Check the motor and associated connections and wiring.
E09	Other Error	The controller may have an internal fault. <ul style="list-style-type: none"> • Check all connections and wiring.

Further issues:

If the above troubleshooting does not help to resolve the issue please contact the authorised dealer. Please have your serial no. to hand to provide to the authorised dealer, which can be found on the battery pack.



11.3 OTHER PROBLEMS

- **Tire:** Low tire pressure: pump up tires to 30~35 psi.
- **Charger:** During charging, light on charger doesn't change to green. Please refer to section 9.

12. WARRANTY

7 YEAR LIMITED WARRANTY
Frame

5 YEAR LIMITED WARRANTY
Motors, Axles & Brakes

1 YEAR LIMITED WARRANTY
Electronics

6 MONTH LIMITED WARRANTY
Battery

The Aspire Zulu scooter comes with a 7-year limited warranty on frame, a 5-year limited warranty on motors, axles, and brakes, a 1-year limited warranty on components, and 6-month on the battery. Please keep purchase invoice as proof of warranty commencement. This product carries a warranty that varies by component part and is against manufacturer defect only, it does not cover wear and tear through use. For further information contact Aidacare.

13. COMPLIANCE

This range of scooters are manufactured in ISO compliant production facilities in Taiwan, adhering to strict quality control standards.

Aspire scooters are manufactured and independently tested to meet relevant market compliance standards.

Labels on the scooter address compliance requirements, and users should familiarise themselves with these before use.

14. RECYCLING INFORMATION

This product is fitted with electronic components and should not be mixed with general household waste. Please follow the diagram below to identify the materials on the product. Old batteries and electronics are to be taken to your nearest recycling centre for recycling or contact your retailer.



Lead Acid Battery



Waste of Electrical and Electronic Equipment (WEEE)



Metal Frame, Plastic Body Parts, and Nylon Upholstery

15. MANUFACTURER, IMPORTER & EU REPRESENTATIVE

If you have any questions, you can contact:




Manufacturer:

Aidacare Pty Ltd ABN 40 134 398 833

Building 3A, 1 Moorebank Avenue,
Moorebank NSW 2170 Australia

1300 133 120 | product@aidacare.com.au

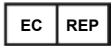

EU Representative:

European Heathcare & Device Solutions (Ireland)Ltd.

Stratton House, Bishopstown Road,
Cork T12 Y9TC Ireland

info@europeandevicesolutions.eu



Importer:

Aidacare Ltd Registration 13785408

Arcadia House, Maritime Walk, Ocean Village,
Southampton SO14 3TL United Kingdom

01622 541235 | info@aidacare.co.uk

The manufacturer reserves the right to alter, without notice, any weights, measurements, or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

AIDACARE PTY LTD

Building 3A, 1 Moorebank Avenue,
Moorebank NSW 2170
Australia

1300 133 120 | aidacare.com.au

MADE IN TAIWAN

070426

